OsteoMed® ExtremiFix

Mid & Large Screw System

EN	Product Information and Instructions for Use	2
FR	Informations sur le produit et instructions d'utilisation.....	7
DE	Produktinformationen und Gebrauchsanweisung ...	13
IT	Informazioni sul prodotto e istruzioni per l'uso	19
PT	Informações sobre o produto e instruções de utilização	24
ES	Información sobre el producto e instrucciones de uso.....	30

Product Information and Instructions for Use

OsteoMed® ExtremiFix Mid & Large Screw System

EN These instructions are intended for the Operating Surgeon and supporting Healthcare Professionals in the European Union only.

Description

OSTEO MED EXTREMI FIX MID & LARGE SCREW SYSTEM is a rigid fixation system comprised of screws in diameters of 4.5mm, 5.5mm and 6.5mm in headed and headless options with various thread types and lengths. The system includes washers for use with the 4.5mm, 5.5mm, and 6.5mm headed screws. The system also includes instruments to facilitate the placement of screws.

Material

OSTEO MED EXTREMI FIX MID & LARGE SCREW SYSTEM: The implants are made from: Titanium Alloy (ASTM F-136). The instrumentation is made from various grades of stainless steel, anodized aluminum, and/or medical grade plastic.

Intended Purpose & Indications for Use

OSTEO MED EXTREMI FIX MID & LARGE SCREW SYSTEM is indicated for use in bone reconstruction, osteotomy, arthrodesis, and fracture fixation of the foot, ankle, and long bones (upper and lower extremity). The screws are intended for single use only. The system drills and guide wires are single use instruments.

The intended clinical benefit of **OSTEO MED EXTREMI FIX MID & LARGE SCREW SYSTEM** is to provide fixation for patients with fractures of bones appropriate for the size of the device as a result of trauma, Osteotomy or reconstruction of the bones. The indirect clinical benefit of the encompassed instrumentation is to facilitate implantation protocols for fracture fixation.

Contraindications

- Active or latent infection or marked inflammation of the treatment area
- Sepsis
- In patients with certain metabolic diseases
- Patients with confirmed material sensitivity
- Patients who are unwilling or incapable of following post-operative care instructions

Warnings

- The treatment or implant may fail, including sudden failure, because of:
 - Loose fixation and/or loosening
 - Stress, including stress from inappropriate bending of the implant during surgery
 - Stress concentrations
 - Stress of weight bearing, load bearing, or excessive activity
 Failure is more likely if the implant experiences increased loads due to delayed union, nonunion, or incomplete healing. Failure is more likely if the patient does not follow post-operative care instructions.
- Nerve or soft tissue damage may result from surgical trauma or the presence of an implant.
- Instrument breakage or damage, as well as tissue damage, may occur when an instrument is subjected to excessive loads, excessive speeds, dense bone, improper use, or unintended use.
- Devices of dissimilar material should not be used together in or near the implant site. Dissimilar metals in contact with each other can accelerate the corrosion process due to galvanic corrosion effects.
- Fractured implants should be removed from patients during surgery. If unable to remove, notify the patient.
- Implants may cause distortion and/or block the view of anatomic structures on radiographic images.

Cautions

- Implants and instruments are intended only for professional use by a licensed physician.
- Use devices in accordance with the Instructions for Use.
- Do not use sterile products past the use-by date. Refer to the device label.
- Do not use or re-sterilize an implant provided in sterile packaging if the package has been damaged. The sterility may be compromised, and the cleanliness of the implant may be uncertain. Report damaged packaging to your distributor or Acumed.
- Use of this system has not been evaluated in children or individuals that are not skeletally mature.
- Inspect all components preoperatively to ensure utility. Do not attempt a surgical procedure with faulty, damaged, or suspect instruments or implants. Alternate fixation methods should be available intraoperatively.
- Reprocessing and/or reuse of single use devices may result in infection/cross contamination, and/or sudden failure due to previous stresses.
- Use of implant components from different manufacturers has not been evaluated.
- Use of chemical disinfection may leave residues that adversely affect steam sterilization.
- Steam penetration and device sterilization may be negatively affected by labeling that block tray steam holes.
- Devices may be contaminated with biohazardous materials and/or sharp materials. Observe hospital procedures, practice guidelines, and/or government regulations for the proper handling of biohazardous material and disposal of sharp materials.

Potential Adverse Effects

- Anesthesia-related problems, problems with positioning of the patient (e.g., nausea, vomiting, neurological impairments, etc.) thrombosis, embolism, infection, or injury to other critical structures such as blood vessels, excessive bleeding, etc.
- Pain, discomfort, or abnormal sensations, nerve or soft tissue damage, necrosis of bone or tissue, bone resorption, or inadequate healing from the presence of an implant or due to surgical trauma.
- Swelling, abnormal scarring, impairment in musculoskeletal function, hardware prominence, migration, loosening, bending, or breakage of the implant, malunion, nonunion or delayed union due to prolonged loading or excessive forces which may lead to implant failure and reoperation.
- Metal sensitivity, histological, allergic, or adverse foreign body reaction resulting from implantation of a foreign material. Consult our document "Metal Sensitivity Statement" at <http://www.acumed.net/ifu>.

- Injury to user.

Note: Any serious incident that has occurred in relation to the device should be reported to Acumed (through <mailto:customercomplaints@acumed.net> or +1.888.627.9957) and the Competent Authority of the Member State in which the user and/or patient is established.

Target Population

The **OSTEOMED EXTREMI-FIX MID & LARGE SCREW SYSTEM** is intended to be used on skeletally mature, adult patients with sufficient quantity and quality of bone who require surgical intervention to achieve fracture fixation, fusion or reconstruction of bones.

MRI Status

Many Acumed implants have been evaluated for safety in the MR environment. Consult our publication "Acumed Implants in the MR Environment" at <http://www.acumed.net/ifu> for more information.

Device Lifetime

Once installed, implants are expected to provide fixation, fusion and physiological support and have an effective life during bone healing. The implants are biocompatible and may remain implanted at the discretion of the surgeon. Multiple-use instruments have a lifetime that is affected by usage, handling, and processing. Assess multiple-use instruments for fitness during pre-sterilization inspection.

Summary of Safety and Clinical Performance (SSCP)

The SSCP for the implants may be obtained from the European database on medical devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>. Use the following search term(s): 08456940BUDIO23NU

Maintaining Device Effectiveness

- The surgeon should have specific training, experience, and thorough familiarity with the use of rigid fixation products and techniques.
- The surgeon must exercise reasonable judgment when deciding which plate and screw type to use for specific indications.
- Devices should be used in an operating room environment.
- All Acumed plates, screws, and instrumentation may be required for each surgery. Failure to use dedicated, unique Acumed instruments for every step of the implantation technique may compromise the integrity of the implanted device, leading to premature device failure that could cause subsequent patient injury. Failed devices may require re-operation and removal.
- Inspect the Acumed implants prior to use. Do not attempt a surgical procedure with faulty, damaged, or suspected instruments or implants. Alternate fixation methods should be available intraoperatively.
- Instrument lifetime is affected by usage, handling, and processing. Assess multiple-use instruments for fitness before and after each procedure to confirm that they are in proper operating condition. Indications that a reusable instrument should not continue to be used include but are not limited to deformation, dulled sharps, cracks, breakage, missing components, corrosion, excessive wear, and restriction or seizure of moving parts. Instruments which are faulty, damaged, or suspected should not be used. They should be replaced or sent to Acumed for disposition and repair.
- When placing more than one screw, ensure that subsequent screw placement does not interfere with other screws. Insert the second screw on the opposite side of the fracture or osteotomy site, and then all remaining screws, following the outlined procedures.

Instructions for Use – OsteoMed ExtremiFix Mid & Large Screw System


- Acumed offers one or more Surgical Technique Guides to promote the safe and effective use of this system. Consult our Surgical Technique Guides at <https://www.acumed.net/resources/documents/>. The Surgical Technique Guides also include information about the installation and calibration of the device.

Important: Surgical Technique Guides may contain important safety information. The instruments and implants in this system are intended to be used by suitably trained and qualified surgeons in a hospital operating room setting. Before treatment, the surgeon is advised to read and fully understand all instructions and communicate to the patient any relevant medical information provided therein, including the use, limitations, risks (safety communications), and possible adverse effects of the proposed treatment. Consult the most recent versions of the Instructions for Use and Surgical Technique Guides as they are subject to change. Contact Acumed or an authorized agent to request any additional information.

Manual Cleaning

Important: Manual Cleaning steps are mandatory. Manual cleaning must be performed prior to performing automated cleaning. Implants are subjected to automated cleaning only.

- Reusable instrumentation must be carefully cleaned prior to sterilization. Trained personnel must perform cleaning and mechanical inspection prior to sterilization.
- Compliance is required with the equipment manufacturer's user instructions (manual and/or machine cleaning, ultrasound treatment, etc.) and recommendations for chemical detergents.
- **Reduction Tool (Midsize 316-1620 & Large 316-1720) Disassembly instructions:**
 1. Press the button down on the body of the Reduction Tool.
 2. While holding the button down, screw the driver clockwise (as viewed from the proximal end) out of the distal end of the body (see illustration):

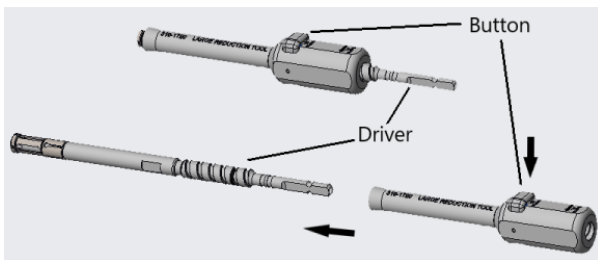- **Reduction Tool Assembly instructions**

1. While holding the button down on the body of the Reduction Tool, screw the driver into the body counter-clockwise (as viewed from the proximal end).
2. Release the button on the body.

The following cleaning instructions are recommended for the reusable instrumentation:

1. Rinse the articles to be cleaned under running cool tap water to remove gross soil.
2. Use a sterile syringe to flush water through any cracks, crevices, lumens, and hard to reach areas. During rinsing, actuate the articles to ensure thorough rinsing.
3. Prepare an enzymatic cleaner such as Enzo[®] per manufacturer's recommendations at 1oz/gal using lukewarm tap water. Using a clean soft cloth that has been soaked in the solution, wipe the entire article. Use a sterile syringe to flush at least 50mL of enzymatic cleaner through any cracks, crevices, lumens, and hard to reach areas. Actuate the articles while immersed to ensure complete penetration of cleaning solution. Allow the articles to soak in the solution for a minimum of 15 minutes.
4. After soaking, thoroughly brush the articles beneath the surface of the prepared cleaner using a soft bristled brush, paying close attention to all hard to reach areas until all evidence of soil is removed. Actuate the articles while brushing in order to clean matted surfaces and movable parts. Using a lumen brush or similar type brush, brush each lumen a minimum of 5 times, ensuring that the lumen brush is passed completely through the entire lumen during brushing. Use a syringe to flush any lumen or matted surface with at least 50mL of solution.

5. Rinse the articles in RO/DI (reverse-osmosis/deionized) water until all visible evidence of detergent is removed. Flush any lumens or matted surfaces with the RO/DI water. Once all evidence of detergent is removed continue to rinse for an additional 30 seconds.
6. Drain excess water from the article and dry using a clean, soft cloth or filtered pressurized air.
7. Prepare a non-enzymatic detergent such as MetriWash™ per manufacturer's recommendations at ¼ oz/gal of lukewarm tap water.
8. Fully immerse the articles and flush all lumens with at least 50mL of the detergent.
9. Allow the articles to soak in the solution for a minimum of 15 minutes.
10. Thoroughly brush the exterior of the articles beneath the surface of the prepared cleaner using a soft bristled brush. Using a lumen brush or similar type brush, brush each lumen a minimum of 5 times, ensuring that the lumen brush is passed completely through the entire lumen during brushing. Flush all lumens with 50mL of solution after brushing.
11. Use a mild detergent such as Metriwash™ and prepare per manufacturer's recommendations of 1/4 oz/gal using lukewarm tap water in a sonication unit. Fully immerse the articles in the prepared detergent. Use a sterile syringe to flush at least 50mL of the prepared detergent through any cracks, crevices, lumens, and hard to reach areas. Actuate the articles while immersed to ensure complete penetration of the cleaning solution. Sonicate the articles for a minimum of 10 minutes.
12. Remove the articles from the sonicator and rinse under RO/DI water for a minimum of 2 minutes. While rinsing, actuate the articles to ensure a thorough rinsing until all visible evidence of detergent is removed. Use a sterile syringe to rinse any lumens or other hard to reach areas with a minimum of 60mL of RO/DI water.
13. Drain excess water and dry the articles using a clean, soft cloth and filtered pressurized air at ≤ 40 psi. Visually inspect each article without magnification for visible soil, deterioration, or loss of function. If soiled, repeat cleaning process. If the device is visibly deteriorated or is unable to function, device shall be returned to Acumed.

Automated Cleaning

Important: Automated cleaning requires manual cleaning. Always perform the previous manual cleaning steps first. Implants are subjected to automated cleaning only. Follow the washer-disinfector manufacturer's instructions explicitly.

1. Rinse the articles to be cleaned under running cool tap water to remove gross soil.
2. Use a sterile syringe to flush water through any cracks, crevices, lumens, and hard to reach areas. During rinsing, actuate the articles to ensure thorough rinsing.
3. Prepare an enzymatic cleaner such as Enzo® per manufacturer's recommendations at 1oz/gal using lukewarm tap water. Using a clean soft cloth that has been soaked in the solution, wipe the entire article. Use a sterile syringe to flush at least 50mL of enzymatic cleaner through any cracks, crevices, lumens, and hard to reach areas. Actuate the articles while immersed to ensure complete penetration of cleaning solution. Allow the articles to soak in the solution for a minimum of 15 minutes.
4. After soaking, thoroughly brush the articles beneath the surface of the prepared cleaner using a soft bristled brush, paying close attention to all hard to reach areas until all evidence of soil is removed. Actuate the articles while brushing in order to clean matted surfaces and movable parts. Using a lumen brush or similar type brush, brush each lumen a minimum of 5 times, ensuring that the lumen brush is passed completely through the entire lumen during brushing. Use a syringe to flush any lumen or matted surface with at least 50mL of solution.
5. Rinse the articles in RO/DI (reverse-osmosis/deionized) water until all visible evidence of detergent is removed. Flush any lumens or matted surfaces with the RO/DI water. Once all evidence of detergent is removed continue to rinse for an additional 30 seconds.
6. Drain excess water from the article and dry using a clean, soft cloth or filtered pressurized air.
7. Prepare a non-enzymatic detergent such as MetriWash™ per manufacturer's recommendations at ¼ oz/gal of lukewarm tap water.
8. Fully immerse the articles and flush all lumens with at least 50mL of the detergent.
9. Allow the articles to soak in the solution for a minimum of 15 minutes.
10. Thoroughly brush the exterior of the articles beneath the surface of the prepared cleaner using a soft bristled brush. Using a lumen brush or similar type brush, brush each lumen a minimum of 5 times, ensuring that the lumen brush is passed completely through the entire lumen during brushing. Flush all lumens with 50mL of solution after brushing.
11. Use a mild detergent such as Metriwash™ and prepare per manufacturer's recommendations of ¼ oz/gal using lukewarm tap water in a sonication unit. Fully immerse the articles in the prepared detergent. Use a sterile syringe to flush at least 50mL of the prepared detergent through any cracks, crevices, lumens, and hard to reach areas. Actuate the articles while immersed to ensure complete penetration of the cleaning solution. Sonicate the articles for a minimum of 10 minutes.
12. Remove the articles from the sonicator and rinse under RO/DI water for a minimum of 2 minutes. While rinsing, actuate the articles to ensure a thorough rinsing until all visible evidence of detergent is removed. Use a sterile syringe to rinse any lumens or other hard to reach areas with a minimum of 60mL of RO/DI water.
13. Drain excess water and dry the articles using a clean, soft cloth and filtered pressurized air at ≤ 40 psi. Visually inspect each article without magnification for visible soil, deterioration, or loss of function. If soiled, repeat cleaning process. If the device is visibly deteriorated or is unable to function, device shall be returned to OsteoMed.

Thermal Disinfection

- Process the devices using a standard washing and thermal disinfection cycle in a washer-disinfector compliant with EN ISO 15883-1 and EN ISO 15883-2 or equivalent national standards.
- Remove all implants if a lubrication phase will be used.
- Process all trays removed from the case (caddy lids remain in place).
- Attach the following instruments to the minimally invasive surgery (MIS) injector or irrigation ports:
 - 320-2800 Ratcheting Driver handle
 - 316-1740 Ratcheting Driver Handle, Large AO
- Thermal disinfection has been validated* for an A0 ≥ 3000 (at least 5 minutes at 90°C).

* Washer-disinfector processing was validated for fully loaded trays with all parts placed appropriately, with recommended evaluations as listed in EN ISO 15883-1 and EN ISO 15883-2 using a STERIS Reliance® Genfore Washer-Disinfector and ENZOL™ Enzymatic Detergent.

Pre-Sterilization Inspection

- Visually inspect all devices under normal lighting to ensure that cleaning was effective. Pay close attention to all challenging areas.
 - Reprocess instruments that are not clean
 - Replace instruments that cannot be cleaned.
- Inspect the implants and instruments for surface damage, such as nicks, scratches, and cracks. Replace affected devices.
- Assess the instruments for proper use. Operate all parts and connecting mechanisms. Give careful attention to drivers, drill bits and reamers, and instruments used for cutting or implant insertion. Critically assess them for wear, sharpness, straightness, and corrosion. Replace any instrument that does not perform as intended.
- Inspect cutting edges under magnification.
 - Replace worn instruments e.g., dull, chipped, cracked, rolled, or otherwise deformed.
 - Run a cotton cloth over the edge to help detect chipping and cracking.
- Verify the legibility of all markings and reference scales. Replace any device that is unreadable.
- Repair, replace, and/or repeat the cleaning of instruments as needed to ensure proper operation before proceeding with sterilization.
- Lubricate (instrument milk) surgical instruments to increase useful life. Do not use silicone-based lubricants, oil, or grease, as these will interfere with steam sterilization. Use a water-based lubricant intended for use on surgical instruments and with steam sterilization. Use the lubricant as directed by the manufacturer. Use critical water if dilution is required.
- Fully replenish the system trays and caddies.

Sterility

- Product is supplied **NON-STERILE unless expressly labeled as STERILE.**
- **DO NOT USE IF THE STERILE PACKAGE IS DAMAGED. DO NOT USE AFTER EXPIRATION DATE.**
- Use of the sterilizer shall comply with the manufacturer’s user instructions for sterilizers.
- Implants and instruments supplied non-sterile must be sterilized prior to use.
- The user facility must clean and dry devices prior to sterilization per standard hospital procedures.
- Follow current industry best practice guidelines such as ANSI/AAMI ST79:2017, the Association for the Advancement of Medical Instrumentation’s (AAMI’s) “Comprehensive guide to steam sterilization and sterility assurance in health care facilities.”
- Non-sterile devices are sterilizable by steam sterilization (autoclaving). For sterilization of Acumed implant systems, the following parameters in the table below should be used. The table shows the minimum parameters validated to achieve a required Sterility Assurance Level (SAL) of 10⁻⁶ for the system.

	<i>The ExtremiFix Mid & Large Screw System - Aluminum Tray, is validated for steam sterilization using one of the below sets of parameters.</i>		<i>The ExtremiFix Mid & Large Screw System - Aluminum Tray, is validated for steam sterilization using one of the below sets of parameters.</i>		<i>The ExtremiFix Mid & Large Screw System - Aluminum Tray, is validated for steam sterilization using one of the below sets of parameters.</i>	
Pre-Vacuum Steam Sterilization	OsteoMed ExtremiFix Mid & Large Screw System – Fully loaded system*		OsteoMed ExtremiFix Mid & Large Screw System – Individual modules		OsteoMed ExtremiFix Mid & Large Screw System – Instrument tray	
Temperature:	132°C	134°C	132°C	134°C	132°C	134°C
Full Cycle Time:	4 min	3 min	4 min	3 min	4 min	3 min
Configuration 1:	Wrapped tray		Wrapped tray		Wrapped tray	
Wrapped Dry Time Minimum:	30 min 15 min Door Open 30 min Outside Chamber	30 min 15 min Door Open 30 min Outside Chamber	30 min 15 min Door Open 30 min Outside Chamber	30 min 15 min Door Open 30 min Outside Chamber	30 min 15 min Door Open 30 min Outside Chamber	30 min 15 min Door Open 30 min Outside Chamber
Wrapped Configuration Details:	<i>Wrap in two layers of 1– ply polypropylene wrap (Kimguard KC600 – 510(k) K082554) using sequential envelope techniques a towel placed between the bottom of the tray and the wraps.</i>		<i>Wrap in two layers of 1– ply polypropylene wrap (Kimguard KC600 – 510(k) K082554) using sequential envelope techniques a towel placed between the bottom of the tray and the wraps.</i>		<i>Wrap in two layers of polypropylene wrap (Haylard Health H600 – 510(k) K082554) using sequential envelope techniques.</i>	
Configuration 2:	Rigid Sterilization Container (RSC)		Rigid Sterilization Container (RSC)		Rigid Sterilization Container (RSC)	
RSC Dry Time Minimum:	20 min 15 min Door Open 30 min Outside Chamber	20 min 15 min Door Open 30 min Outside Chamber	20 min 15 min Door Open 30 min Outside Chamber	20 min 15 min Door Open 30 min Outside Chamber	20 min 15 min Door Open 30 min Outside Chamber	20 min 15 min Door Open 30 min Outside Chamber
RSC Configuration Details:	<i>Two wide-body rigid sterilization containers (such as Aesculap JN442), each with a perforated basket (such as Aesculap JF101R) should be used for RSC sterilization.</i>		<i>A wide-body rigid sterilization container (such as Aesculap JN442), with a perforated basket (such as Aesculap JF101R) should be used for RSC sterilization.</i>		<i>A wide-body rigid sterilization container (such as Aesculap JN442), with a perforated basket (such as Aesculap JF101R) should be used for RSC sterilization.</i>	
<i>Note: Biological indicator, G. stearothermophilus, was used in sterilization validation.</i>						

Post-Sterilization Inspection

- Do not store or use sterile devices if they are not dry.
- Inspect the sterile barrier for signs of damage. Do not use the product if the sterile barrier has been compromised.

Storage

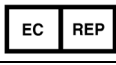
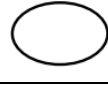
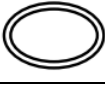
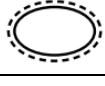
- Store sterilized devices under controlled conditions in a manner that minimizes the potential for contamination per ANSI/AAMI ST79:2017. Refer to sterilization wrap or rigid container manufacturer’s IFU for limits on sterile product storage time and storage requirements for temperature and humidity.
- Devices should be stored in an area that provides protection from dust, pests, and temperature/ humidity extremes.

Safe Disposal

Devices may be contaminated with biohazardous materials and/or sharp materials. Observe hospital procedures, practice guidelines, and/or government regulations for the proper handling biohazardous material and disposal of sharp materials.

Symbols and Definitions

Note: This is a general list of symbols. Not all symbols apply to all systems.

Symbol	Description	ISO 15223-1	Symbol	Description	ISO 15223-1
	Manufacturer	5.1.1		Authorized Representative in the European Community	5.1.2
	Date of Manufacture	5.1.3		Use-by Date	5.1.4
	Batch Code (Lot Number)	5.1.5		Catalogue Number	5.1.6
	Serial number	5.1.7		Sterilized using irradiation	5.2.4
	Do Not Resterilize	5.2.6		Non-Sterile	5.2.7
	Do not use if sterile package is damaged	5.2.8		Single Sterile Barrier System	5.2.11
	Double Sterile Barrier System	5.2.12		Single Sterile Barrier System with Protective packaging inside	5.2.13
	Single Sterile Barrier System with Protective packaging inside	5.2.14		Single Use Only	5.4.2
 www.acumed.net/ifu	Consult Instructions for Use	5.4.3	Rx only	Caution: U.S. federal law restricts this device to sale by or on the order of a physician.	U.S. 21 CFR 801.109
	Medical Device	5.7.7		MR Conditional (ASTM F2503)	
	CE marking of conformity, Article 20 of Regulation (EU) 2017/745. CE marking may be accompanied by the identification number of the notified body responsible for conformity assessment.				

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CUSTOMER SERVICE (OUTSIDE USA):

+1.888.627.9957

These materials contain information about products that may or may not be available in any particular country or may be available under different trademarks in different countries. The products may be approved or cleared by governmental regulatory organizations for sale or use with different indications or restrictions in different countries. Products may not be approved for use in all countries. Nothing contained in these materials should be construed as a promotion or solicitation for any product or for the use of any product in a particular way that is not authorized under the laws and regulations of the country where the reader is located. Nothing in these materials should be construed as a representation or warranty as to the efficacy or quality of any product, nor the appropriateness of any product to treat any specific condition. Physicians may direct questions about the availability and use of the products described in these materials to their authorized Acumed distributor. Specific questions patients may have about the use of the products described in these materials or the appropriateness for their own conditions should be directed to their own physician.

PKGI-014295 Rev B | 08-2025 | © 2024 Acumed® LLC

OsteoMed® is a registered trademark of Acumed, LLC.

Subsidiaries:

Acumed Ltd

Huebner House

The Fairground

Andover

Hampshire UK SP11 0QN

Tel: +44 1264 774450

Acumed Iberica

C/ Procion 1

Edificio Oficor

28023 Madrid-Spain

Tel: +34 913516357

Acumed GmbH

Fuhlsbuttler Stasse 300

22307 Hamburg

Deutschland

Tel: +49-40 947 82 093

EMERGO EUROPE

WESTERVOORTSEDIJK 60

6827 AT ARNHEM

THE NETHERLANDS

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ACUMED, LLC