ACUMED®

ACUMED SINGLE USE INSTRUMENTS

ACUMED SINGLE USE TAPPING INSTRUMENTS

MediMark® Europe Sarl.
11 rue Emile ZOLA. BP 2332
38033 GRENOBLE CEDEX 2
FRANCE
+33 4 76 86 43 22

Acumed® LLC
5885 NW Cornelius Pass Road
Hillsboro, OR 97124-9432
+1 (503) 627-9957
www.acumed.net

PKGI-52-E
Effective 4/2015

ACUMED-INSTRUMENTER TIL ENGANGSBRUG, ACUMED BANKE-LET INSTRUMENTER TIL ENGANGSBRUG

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ACUMED SINGLE USE INSTRUMENTS, ACUMED SINGLE USE TAPPING INSTRUMENTS

FOR THE PERSONAL ATTENTION OF THE OPERATING SURGEON

DESCRIPTION: Acumed Disposable, Single Use Surgical Instruments are available for a wide variety of surgical techniques and implant systems. Acumed Disposable, Single Use Surgical Tap Instruments are only available for use with Acumed bone plates. These instruments should be discarded after each use.

WARNINGS: For safe effective use of any Acumed instrument, the surgeon must be thoroughly familiar with the instrument, the method of application, and the recommended surgical technique. Instrument breakage or damage may occur when the instrument is subjected to excessive loads, speeds, or dense bone. The surgical tap should only be used with Acumed bone plates as recommended in the surgical technique. Use with plates or implants not recommended in the surgical technique can cause plate, tap, and/or treatment failure. The patient must be cautioned, preferably in writing, about the risks associated with these types of instruments.

PRECAUTIONS: A Single Use Surgical Instrument shall never be reused. Previous stresses may have created imperfections, which can lead to a device failure. Protect instruments against scratching and nicking, such stress concentrations can lead to failure.

Tapping a plate using a plate tap will cause titanium debris to be generated, which should be removed. Failure to remove the plate debris can cause, among other complications, inflammation, cartilage damage, and patient discomfort. The taps are single surgery use and should be discarded after each surgery or if the tap becomes dull or damaged. If the resistance increases while using a tap, discard the tap immediately. Breakage to the tap can occur due to excessive torque or levering and care should be taken to avoid such conditions. Should breakage occur, carefully remove all tap pieces.

CLEANING INSTRUCTIONS: Acumed Single-Use Instruments should never be reused. However, Acumed Single-Use Instruments designed for storage within a reusable instrument tray, may remain placed in the tray during cleaning processes for the reusable components. Acumed recommendations for cleaning reusable devices are:

Acumed Instruments and Accessories must be thoroughly cleaned before reuse. Decontamination of reusable instruments or accessories should occur immediately after completion of the surgical procedure. Excess blood or debris should be wiped off to prevent it from drying onto the surface. Using an enzymatic cleaning product such as Enzol, prepare the cleaning solution according to manufacturer's instructions. Avoiding prolonged soaking, submerge the device in the enzymatic cleaning solution for 5 minutes. After 5 minutes, use a soft bristle brush to remove any detergent residue. Instruments with lumens also should be held under water (larger side up) ensuring the lumen is flushed with water. Place the device in an ultrasonic cleaner filled with deionized water ensuring that there is no air trapped in any of the devices by gently agitating the device. Sonicate the device for 5 minutes. Remove the device from the ultrasonic cleaner and rinse in water by submerging it completely. Gently wipe the device with a clean, dry cloth, then air dry.

Even surgical instruments manufactured from high grade stainless steel must be dried thoroughly to prevent rust formation, and all devices must be inspected for cleanliness of surfaces, joints, and lumens, proper function, and wear and tear prior to sterilization.

SPECIAL CLEANING REQUIREMENTS:

Exposed springs, coils, or flexible features: Flood the crevices with copious amounts of cleaning solution to flush out any soil. Scrub the surface with a scrub brush to remove all visible soil from the surface and crevices. Bend the flexible area and scrub the surface with a scrub brush. Rotate the part while scrubbing to ensure that all crevices are cleaned.

ANODIZED ALUMINUM: Anodized aluminum must not come in contact with certain cleaning or disinfectant solutions. Avoid strong alkaline cleaners and disinfectants or solutions containing iodine, chlorine or certain metal salts. Also, in solutions with pH values above 11, the anodization layer may dissolve.

TORQUE LIMITING DRIVERS: To prolong the life of torque limiting drivers,

moving parts must be lubricated with a commercial water-based surgical grade instrument lubricant (instrument milk) after every use. Lubrication should be performed after cleaning and prior to sterilization. Follow lubricant manufacturer's instructions.

STERILITY: These products are provided non-sterile. Sterilization may be performed using one of the following methods.

Sterilization Methods

Gravity Displacement Autoclave	Full Cycle Time: 270° F (132° C) for 60 minutes Dry Time: 125 minutes
Pre-Vacuum Autoclave	Full Cycle Time: 270° F (132° C) for 25 minutes Dry Time: 70 minutes

- Consult your equipment manufacturer's written instructions for specific sterilizer and load configuration instructions.
- Follow current AORN® "Recommended Practices for Sterilization in Perioperative Practice Settings" and ANSI/AAMI ST79: 2010 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79: 2010 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

STORAGE INSTRUCTIONS: Store in a cool dry place and keep away from direct sunlight. Prior to use, inspect product package for signs of tampering, or water contamination. Use oldest lots first.

CAUTION: For Professional Use Only.

CAUTION: Federal Law (USA) restricts this product sale by or on the order of a physician or hospital.

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Acumed Instruments and Accessories must be thoroughly cleaned before reuse. Decontamination of reusable instruments or accessories should occur immediately after completion of the surgical procedure. Excess blood or debris should be wiped off to prevent it from drying onto the surface. Using an enzymatic cleaning product such as Enzol, prepare the cleaning solution according to manufacturer's instructions. Avoiding prolonged soaking, submerge the device in the enzymatic cleaning solution for 5 minutes. After 5 minutes, use a soft bristle brush to remove any detergent residue. Instruments with lumens also should be held under water (larger side up) ensuring the lumen is flushed with water. Place the device in an ultrasonic cleaner filled with deionized water, ensuring that there is no air trapped in any of the crevices by gently agitating the device. Soak the device for 5 minutes. Remove the device from the ultrasonic cleaner and rinse in water by submerging it completely. Gently wipe the device with a clean, dry cloth, then air dry.

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Anodized aluminum: Anodized aluminum must not come in contact with certain cleaning or disinfectant solutions. Avoid strong alkaline cleaners and disinfectants or solutions containing iodine, chlorine or certain metal salts. Also, in solutions with pH values above 11, the anodization layer may dissolve.

Torque limiting drivers: To prolong the life of torque limiting drivers, moving parts must be lubricated with a commercial water-based surgical grade instrument lubricant (instrument milk) after every use. Lubrication should be performed after cleaning and prior to sterilization. Follow lubricant manufacturer's instructions.

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- Consult your equipment manufacturer's written instructions for specific sterilizer and load configuration instructions.
- Follow current AORN "Recommended Practices for Sterilization in Perioperative Practice Settings" and ANSI/AAMI ST79:2010 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79:2010 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

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