ACUMED®

ACUMED SINGLE USE INSTRUMENTS

ACUMED SINGLE USE TAPPING INSTRUMENTS

MediMark® Europe Sarl.

11 rue Emile ZOLA. BP 2332

38033 GRENOBLE CEDEX 2

FRANCE

+33 4 76 86 43 22

Acumed®

5885 NW Cornelius Pass Road

Hillsboro, OR 97124-9432

+1 (503) 627-9957

www.acumed.net

PKGI-57-A

Effective 01/2011

* P K G I - 5 7 - A *

OBSOLETE
Visit www.acumed.net/ifu
for the latest version.

ACUMED-INSTRUMENTER TIL ENGANGSBRUG, ACUMED BANKE-LET INSTRUMENTER TIL ENGANGSBRUG

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WARNINGS: For safe effective use of any Acumed instrument, the surgeon must be thoroughly familiar with the instrument, the method of application, and the recommended surgical technique. Instrument breakage or damage may occur when the instrument is subjected to excessive loads, speeds, or dense bone. The surgical tap should only be used with Acumed bone plates as recommended in the surgical technique. Use with plates or implants not recommended in the surgical technique can cause plate, tap, and/or treatment failure. The patient must be cautioned, preferably in writing, about the risks associated with these types of instruments.

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Sterilization Methods

Gravity Displacement Autoclave	NOT RECOMMENDED
Pre-Vacuum Autoclave	Full cycle Time: 132° C (270° F) for 4 minutes Dry Time: 30 minutes

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US

ACUMED SINGLE USE INSTRUMENTS, ACUMED SINGLE USE TAPPING INSTRUMENTS

FOR THE PERSONAL ATTENTION OF THE OPERATING SURGEON

DESCRIPTION: Acumed Disposable, Single Use Surgical Instruments are available for a wide variety of surgical techniques and implant systems. Acumed Disposable, Single Use Surgical Tap Instruments are only available for use with Acumed bone plates. These instruments should be discarded after each use.

WARNINGS: For safe effective use of any Acumed instrument, the surgeon must be thoroughly familiar with the instrument, the method of application, and the recommended surgical technique. Instrument breakage or damage may occur when the instrument is subjected to excessive loads, speeds, or dense bone. The surgical tap should only be used with Acumed bone plates as recommended in the surgical technique. Use with plates or implants not recommended in the surgical technique can cause plate, tap, and/or treatment failure. The patient must be cautioned, preferably in writing, about the risks associated with these types of instruments. The implant system has not been evaluated for safety and compatibility in the MR (Magnetic Resonance) environment, nor has it been tested for heating or migration in the MR environment.

PRECAUTIONS: A Single Use Surgical Instrument shall never be reused. Previous stresses may have created imperfections, which can lead to a device failure. Protect instruments against scratching and nicking, such stress concentrations can lead to failure.

Tapping a plate using a plate tap will cause titanium debris to be generated, which should be removed. Failure to remove the plate debris can cause, among other complications, inflammation, cartilage damage, and patient discomfort. The taps are single surgery use and should be discarded after each surgery or if the tap becomes dull or damaged. If the resistance increases while using a tap, discard the tap immediately. Breakage to the tap can occur due to excessive torque or levering and care should be taken to avoid such conditions. Should breakage occur, carefully remove all tap pieces.

STERILITY: These products are provided non-sterile. Sterilization may be performed using one of the following methods.

Sterilization Methods

Gravity Displacement Autoclave	NOT RECOMMENDED
Pre-Vacuum Autoclave	Full Cycle Time: 270° F (132° C) 104 minutes Dry Time: 30 minutes

- Consult your equipment manufacturer's written instructions for specific sterilizer and load configuration instructions.
- Follow current AORN "Recommended Practices for Sterilization in Perioperative Practice Settings" and ANSI/AAMI ST79: 2010 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79: 2010 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

STORAGE INSTRUCTIONS: Store in a cool, dry place and keep away from direct sunlight. Prior to use, inspect product package for signs of tampering, or water contamination. Use oldest lots first.

CAUTION: For Professional Use Only.

CAUTION: Federal Law (USA) restricts this product sale by or on the order of a physician or hospital.