OsteoMed® ExtremiLock

Wrist Plating System

EN	Product Information and Instructions for Use	2
FR	Informations sur le produit et instructions d'utilisation.....	6
DE	Produktinformationen und Gebrauchsanweisung ...	10
IT	Informazioni sul prodotto e istruzioni per l'uso	14
PT	Informações sobre o produto e instruções de utilização	18
ES	Información sobre el producto e instrucciones de uso.....	22

Product Information and Instructions for Use

OsteoMed® ExtremiLock Wrist Plating System

EN

These instructions are intended for the Operating Surgeon and supporting Healthcare Professionals in the European Union only.

Description

The **OsteoMed ExtremiLock Wrist Plating System** is a rigid fixation system consisting of plates and screws in various configurations. Plates are provided in a variety of shapes and sizes and offer surgeons positioning and locking hole designs. The **OsteoMed ExtremiLock Wrist Plating System** includes variable angled locking, non-locking, fully threaded, partially threaded, and cannulated screws, washers, as well as K-wires. Surgical instrumentation is provided to facilitate modification and insertion of the implants. The variable locking angle range for the 2.4mm screws is ± 15 degrees and for the 2.7mm screws it is ± 10 degrees.

Material

The plates are made from titanium (ASTM F-67). Screws are made from titanium alloy (ASTM F-136). The instrumentation is made from various grades of stainless steel, titanium alloy, anodized aluminum, and/or medical grade polymers. The material composition of Acumed implants is described in detail in the available Metal Sensitivity Statement document "Metal Sensitivity Statement" which can be found at www.acumed.net/ifu.

Intended Purpose

The **OsteoMed ExtremiLock Wrist Plating System** is intended for fracture fixation, fusion osteotomies of the wrist and other bones that are appropriate for their size of the device. The **OsteoMed ExtremiLock Wrist Plating System** implants are intended for single use only.

The intended clinical benefit of **OsteoMed ExtremiLock Wrist Plating System** is to provide fixation for patients with fractures as a result of trauma, Osteotomy or reconstruction of the bones. The indirect clinical benefit of the encompassed instrumentation is to facilitate implantation protocols for fracture fixation.

Indications for use

The **OsteoMed ExtremiLock Wrist Plating System** is indicated for fracture fixation, fusion, and osteotomies of wrist and other bones appropriate for the size of the device. It is intended for use in trauma, general surgery, and reconstructive procedures.

Contraindications

- Active or latent infection or marked inflammation of the treatment area
- Sepsis
- Insufficient quantity or quality of bone, osteoporosis, or in patients with certain metabolic diseases
- Patients with confirmed material sensitivity
- Patients who are unwilling or incapable of following post-operative care instructions and/or the limitations of internal rigid fixation implants

Warnings

- The treatment or implant may fail, including sudden failure, because of:
 - Loose fixation and/or loosening
 - Stress, include stress from inappropriate bending of the implant during surgery
 - Stress concentrations
 - Stress of wearing, load bearing, or excessive activity
 Failure is more likely if the implant experiences increased loads due to delayed union, nonunion, or incomplete healing. Failure is more likely if the patient does not follow post-operative care instructions.
- Nerve or soft tissue damage may result from surgical trauma or the presence of an implant.
- Instrument breakage or damage, as well as tissue damage, may occur when an instrument is subjected to excessive loads, excessive speeds, dense bone, improper use, or unintended use.
- Devices of dissimilar material should not be used together in or near the implant site. Dissimilar metals in contact with each other can accelerate the corrosion process due to galvanic corrosion effects.
- Fractured implants should be removed from patients during surgery. If unable to remove, notify the patient.
- Implants may cause distortion and/or block the view of anatomic structures on radiographic images.
- In patients with a large intramedullary canal, the diameter/length of the Fusion screw provided may not provide adequate compression of the MCP joint.

Cautions

- Implants and instruments are intended only for professional use by a licensed physician.
- Use devices and instruments in accordance with the Instructions for Use.
- Do not use the sterile product past the use-by date. Refer to the device label.
- Do not use or re-sterilize an implant provided in sterile packaging if the package has been damaged. The sterility may be compromised, and the cleanliness of the implant may be uncertain. Report damaged packaging to your distributor or Acumed.
- Use of this system has not been evaluated in children or individuals that are not skeletally mature.
- Inspect all components preoperatively to ensure utility. Do not attempt a surgical procedure with faulty, damaged, or suspect instruments or implants. Alternate fixation methods should be available intraoperatively.
- Reprocessing and/or reuse of single use devices may result in infection/cross contamination, and/or sudden failure due to previous stresses.
- Use of implant components from different manufacturers have not been evaluated.
- Use of chemical disinfection may leave residues that adversely affect steam sterilization.
- Steam penetration and device sterilization may be negatively affected by labeling that block tray steam holes.
- Devices may be contaminated with biohazardous materials and/or sharp materials. Observe hospital procedures, practice guidelines, and/or government regulations for the proper handling biohazardous material and disposal of sharp materials.

Potential Adverse Effects

- Anesthesia-related problems, problems with positioning of the patient (e.g., nausea, vomiting, neurological impairments, etc.) thrombosis, embolism, infection, or injury to other critical structures such as blood vessels, excessive bleeding, etc.
- Pain, discomfort, or abnormal sensations, nerve or soft tissue damage, necrosis of bone or tissue, bone resorption, or inadequate healing from the presence of an implant or due to surgical trauma.
- Swelling, abnormal scarring, impairment in musculoskeletal function, hardware prominence, migration, loosening, bending, or breakage of the implant, malunion, nonunion or delayed union due to prolonged loading or excessive forces which may lead to implant failure and reoperation.

- Metal sensitivity, histological, allergic, or adverse foreign body reaction resulting from implantation of a foreign material. Consult our document “Metal Sensitivity Statement” at <http://www.acumed.net/ifu>.
- Injury to user.

Important: Any serious incident that has occurred in relation to the device should be reported to Acumed (through customercomplaints@acumed.net or +1.888.627.9957) and the Competent Authority of the Member State in which the user and/or patient is established.

Target Population

The system is intended for use in skeletally mature, adult patients who suffer from fractures or osteotomies of the wrist and who need general surgical intervention for wrist trauma or reconstructive surgery.

MRI Status

Many Acumed implants have been evaluated for safety in the MR environment. Consult our publication “Acumed Implants in the MR Environment” at www.acumed.net/ifu for more information.

Device Lifetime

Once installed, implants are expected to provide fixation, fusion and physiological support and have an effective life during bone healing. The implants are biocompatible and may remain implanted at the discretion of the surgeon. Multiple-use instruments have a lifetime that is affected by usage, handling, and processing. Assess multiple-use instruments for fitness during pre-sterilization inspection.

Summary of Safety and Clinical Performance (SSCP)

The SSCP for the implants may be obtained from the European database on medical devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>. Use the following search terms: 08456940BUDIO24NW.

Note: Any serious incident that has occurred in relation to the device should be reported to Acumed (through customercomplaints@acumed.net or +1.888.627.9957) and the competent authority of the Member State in which the user and/or patient is established.

Maintaining Device Effectiveness

- The surgeon should have specific training, experience, and thorough familiarity with the use of rigid fixation products and techniques.
- The surgeon must exercise reasonable judgment when deciding which plate and screw type to use for specific indications.
- Devices should be used in an operating room environment.
- All Acumed plates, screws, and instrumentation may be required for each surgery. Failure to use dedicated, unique Acumed instruments for every step of the implantation technique may compromise the integrity of the implanted device, leading to premature device failure that could cause subsequent patient injury. Failed devices may require re-operation and removal.
- Inspect the Acumed implants prior to use. Do not attempt a surgical procedure with faulty, damaged, or suspected instruments or implants. Alternate fixation methods should be available intraoperatively.
- Instrument lifetime is affected by usage, handling, and processing. Assess multiple-use instruments for fitness before and after each procedure to confirm that they are in proper operating condition. Indications that a reusable instrument should not continue to be used include but are not limited to deformation, dulled sharps, cracks, breakage, missing components, corrosion, excessive wear, and restriction or seizure of moving parts. Instruments which are faulty, damaged, or suspected should not be used. They should be replaced or sent to Acumed for disposition and repair.
- When placing more than one screw, ensure that subsequent screw placement does not interfere with other screws. Insert the second screw on the opposite side of the fracture or osteotomy site, and then all remaining screws, following the outlined procedures.

Instructions for Use – OsteoMed ExtremiLock Wrist Plating System

- Acumed offers one or more Surgical Technique Guides to promote the safe and effective use of this system. Consult our Surgical Technique Guides at <http://www.acumed.net/ifu>. The Surgical Technique Guides also include information about the installation and calibration of the device.
- **Important:** Surgical Technique Guides may contain important safety information. The instruments and implants in this system are intended to be used by suitably trained and qualified surgeons in a hospital operating room setting. Before treatment, the surgeon is advised to read and fully understand all instructions and communicate to the patient any relevant medical information provided therein, including the use, limitations, risks (safety communications), and possible adverse effects of the proposed treatment. Consult the most recent versions of the Instructions for Use and Surgical Technique Guides as they are subject to change. Contact Acumed or an authorized agent for additional information.

Cleaning

1. Rinse the articles to be cleaned under running cool tap water to remove gross soil.
2. Use a sterile syringe to flush water through any cracks, crevices, lumens, and hard to reach areas. During rinsing, actuate the articles to ensure thorough rinsing.
3. Prepare an enzymatic cleaner such as Enzo[®] per manufacturer’s recommendations at 1oz/gal using lukewarm tap water. Using a clean soft cloth that has been soaked in the solution, wipe the entire article. Use a sterile syringe to flush at least 50mL of enzymatic cleaner through any cracks, crevices, lumens, and hard to reach areas. Actuate the articles while immersed to ensure complete penetration of cleaning solution. Allow the articles to soak in the solution for a minimum of 15 minutes.
4. After soaking, thoroughly brush the articles beneath the surface of the prepared cleaner using a soft bristled brush, paying close attention to all hard to reach areas until all evidence of soil is removed. Actuate the articles while brushing in order to clean matted surfaces and movable parts. Using a lumen brush or similar type brush, brush each lumen a minimum of 5 times, ensuring that the lumen brush is passed completely through the entire lumen during brushing. Use a syringe to flush any lumen or matted surface with at least 50mL of solution.
5. Rinse the articles in RO/DI (reverse-osmosis/deionized) water until all visible evidence of detergent is removed. Flush any lumens or matted surfaces with the RO/DI water. Once all evidence of detergent is removed continue to rinse for an additional 30 seconds.
6. Drain excess water from the article and dry using a clean soft cloth or filtered pressurized air.
7. Prepare a non-enzymatic detergent such as MetriWash[™] per manufacturer’s recommendations at ¼ oz/gal of lukewarm tap water.
8. Fully immerse the articles and flush all lumens with at least 50mL of the detergent.
9. Allow the articles to soak in the solution for a minimum of 15 minutes.
10. Thoroughly brush the exterior of the articles beneath the surface of the prepared cleaner using a soft bristled brush. Using a lumen brush or similar type brush, brush each lumen a minimum of 5 times, ensuring that the lumen brush is passed completely through the entire lumen during brushing. Flush all lumens with 50mL of solution after brushing.
11. Use a mild detergent such as MetriWash[™] and prepare per manufacturer’s recommendations of ¼ oz/gal using lukewarm tap water in a sonication unit. Fully immerse the articles in the prepared detergent. Use a sterile syringe to flush at least 50mL of the prepared detergent through any cracks, crevices, lumens, and hard to reach areas. Actuate the articles while immersed to ensure complete penetration of the cleaning solution. Sonicate the articles for a minimum of 10 minutes.
12. Remove the articles from the sonicator and rinse under RO/DI water for a minimum of 2 minutes. While rinsing, actuate the articles to ensure a thorough rinsing until all visible evidence of detergent is removed. Use a sterile syringe to rinse any lumens or other hard to reach areas with a minimum of 60mL of RO/DI water.
13. Drain excess water and dry the articles using a clean, soft cloth and filtered pressurized air at ≤ 40 psi. Visually inspect each article without magnification for visible soil, deterioration, or loss of function. If soiled, repeat cleaning process. If device is visibly deteriorated or is unable to function, device shall be returned to OsteoMed.

Thermal Disinfection

Utilize a thermal disinfection program consistent with A0 value > 3000 or perform thermal disinfection at 90°C (maximum temperature 95°C) for at least 5 minutes.

Pre-Sterilization Inspection

- Visually inspect all devices under normal lighting to ensure that cleaning was effective. Pay close attention to all challenging areas. Reprocess instruments that are not clean and replace instruments that cannot be cleaned.
- Inspect the implants and the instruments for surface damage, such as nicks, scratches, and cracks. Replace affected devices.
- Assess the instruments for proper use. Operate all parts and connecting mechanisms. Give careful attention to drivers, drill bits and reamers, and instruments used for cutting or implant insertion. Critically assess them for wear, sharpness, straightness, and corrosion. Replace any instrument that does not perform as intended.
- Inspect cutting edges under magnification. Replace worn instruments e.g., dull, chipped, cracked, rolled, or otherwise deformed. Run a cotton cloth over the edge to help detect chipping and cracking.

- Verify the legibility of all markings and reference scales. Replace any device that is unreadable.
- Repair, replace, and/or repeat the cleaning of instruments as needed to ensure proper operation before proceeding with sterilization.
- Lubricate (instrument milk) surgical instruments to increase useful life. Do not use silicone-based lubricants, oil, or grease, as these will interfere with steam sterilization. Use a water-based lubricant intended for use on surgical instruments and with steam sterilization. Use the lubricant as directed by the manufacturer. Use critical water if dilution is required.
- Fully replenish the system trays and caddies.

Sterility

- Product is supplied **NON-STERILE unless expressly labeled as STERILE.**
- **DO NOT USE IF THE STERILE PACKAGE IS DAMAGED. DO NOT USE AFTER EXPIRATION DATE.**
- Use of the sterilizer shall comply with the manufacturer’s user instructions for sterilizers.
- Implants and instruments supplied non-sterile must be sterilized prior to use.
- The user facility must clean and dry devices prior to sterilization per standard hospital procedures. Follow current industry best practice guidelines such as ANSI/AAMI ST79:2017, the Association for the Advancement of Medical Instrumentation's (AAMI's) "Comprehensive guide to steam sterilization and sterility assurance in health care facilities."
- Non-sterile devices are sterilizable by steam sterilization (autoclaving). For sterilization of Acumed implant systems, the following parameters in the table below should be used. The table shows the minimum parameters validated to achieve a required Sterility Assurance Level (SAL) of 10-6 for the system.

The OsteoMed ExtremiLOCK Wrist System Aluminum Tray, is validated for steam sterilization using one of the below sets of parameters.		
Pre-Vacuum Steam Sterilization	ExtremiLOCK Wrist System Tray Fully loaded system (320-6309)*	
Temperature:	132°C	134°C
Full Cycle Time:	4 minutes	3 minutes
Configuration 1:	Wrapped tray	
Wrapped Dry Time Minimum:	30 minutes	40 minutes
Wrapped Configuration Details:	Wrap in two layers of 1- ply polypropylene wrap (Kimguard KC600 – 510(k) K082554) using sequential envelope techniques.	
Configuration 2:	Rigid Sterilization Container**	
Rigid Container Dry Time Minimum:	30 minutes	30 minutes
*The ExtremiLOCK Wrist Plating System can include 4 of the 6 following modules: 320-6301, 320-6302, 320-6303, 320-6306, 320-6307, 320-6308		
**OsteoMed recommends using Aesculap rigid containers & paper filters. A single-level ExtremiLOCK Wrist Plating System tray and contents may be sterilized in an Aesculap container base P/N JN446 with lid P/N JK489. Use 4 Aesculap paper filters P/N US751, 2 in the lid and 2 in the bottom of the rigid container.		
Note: Biological indicator, G. stearothermophilus, was used in sterilization validation.		

Storage

- Store sterilized devices under controlled conditions in a manner that minimizes the potential for contamination per ANSI/AAMI ST79:2017. Refer to sterilization wrap or rigid container manufacturer’s IFU for limits on sterile product storage time and storage requirements for temperature and humidity.
- Devices should be stored in an area that provides protection from dust, pests, and temperature/ humidity extremes.

Safe Disposal

Devices may be contaminated with biohazardous materials and/or sharp materials. Observe hospital procedures, practice guidelines, and/or government regulations for the proper handling biohazardous material and disposal of sharp materials.

Symbols and Definitions

Note: This is a general list of symbols. Not all symbols apply to all systems.

Symbol	Description	ISO 15223-1	Symbol	Description	ISO 15223-1
	Manufacturer	5.1.1		Authorized Representative in the European Community	5.1.2
	Date of Manufacture	5.1.3		Use-by Date	5.1.4
	Batch Code (Lot Number)	5.1.5		Catalogue Number	5.1.6
	Serial number	5.1.7		Sterilized using irradiation	5.2.4
	Do Not Resterilize	5.2.6		Non-Sterile	5.2.7
	Single Sterile Barrier System	5.2.11		Single Sterile Barrier System with Protective packaging inside	5.2.14
	Double Sterile Barrier System	5.2.12		Single Sterile Barrier System with Protective packaging inside	5.2.13
 www.acumed.net/ifu	Consult Instructions for Use	5.4.3	Rx only	Caution: U.S. federal law restricts this device to sale by or on the order of a physician.	U.S. 21 CFR 801.109
	Do not use if package is damaged or opened and consult Instructions for Use	5.2.8		Do not re-use	5.4.2
	Medical Device	5.7.7		MR Conditional (ASTM F2503)	
	CE marking of conformity, Article 17 of EU Directive 93/42/EEC or Article 20 of Regulation (EU) 2017/745. CE marking may be accompanied by the identification number of the notified body responsible for conformity assessment				

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CUSTOMER SERVICE (OUTSIDE USA):

+1.888.627.9957

These materials contain information about products that may or may not be available in any particular country or may be available under different trademarks in different countries. The products may be approved or cleared by governmental regulatory organizations for sale or use with different indications or restrictions in different countries. Products may not be approved for use in all countries. Nothing contained in these materials should be construed as a promotion or solicitation for any product or for the use of any product in a particular way that is not authorized under the laws and regulations of the country where the reader is located. Nothing in these materials should be construed as a representation or warranty as to the efficacy or quality of any product, nor the appropriateness of any product to treat any specific condition. Physicians may direct questions about the availability and use of the products described in these materials to their authorized Acumed distributor. Specific questions patients may have about the use of the products described in these materials or the appropriateness for their own conditions should be directed to their own physician.

PKGI-014292-Rev A | 11-2024 | © 2024 Acumed® LLC

OsteoMed® is a registered trademark of Acumed, LLC.

Subsidiaries:

Acumed Ltd

Huebner House

The Fairground

Andover

Hampshire UK SP11 0QN

Tel: +44 1264 774450

Acumed Iberica

C/ Procion 1

Edificio Oficor

28023 Madrid-Spain

Tel: +34 913516357

Acumed GmbH

Fuhlsbuttler Stasse 300

22307 Hamburg

Deutschland

Tel: +49-40 947 82 093

EMERGO EUROPE

WESTERVOORTSEDIJK 60

6827 AT ARNHEM

THE NETHERLANDS

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ACUMED, LLC